H440E  
H440M  
H450L - H450N  
H450L-I  
H450LS  
H460B  
H460T  
H460V  
H461L  
(+ H441)  
H450LA  
H750L  
H407L

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Bruksanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

## I. FRANCAIS

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## 2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

**The following operating and maintenance instructions must be observed :**

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

### FOR ALL COMMODES EXCEPT H407L

- The structural components are assembled and retained using plastic fixings. In any case the structure should not be delivered in disassembled form. You should not attempt to disassemble or re-assemble this yourself.
- Ensure that the commode is resting on a flat, non-slip surface before use.
- The perforated seat can be removed to facilitate cleaning. Ensure that the perforated seat is firmly fitted to the commode before use (the locking points are marked).
- The commodes are designed for indoor use only and should not be used as shower seats.

**H440E, H440M, H450L, H450N, H450L-I, H450LS, H460B, H460T, H460V AND H461L OMEGA FIXED COMMODES**

Do not attempt to move the commode while someone is sitting on it.

### H450LA OMEGA HEIGHT-ADJUSTABLE COMMODE

- Adjust the leg height using the spring buttons. A maximum difference of one hole forward or back is permitted. The right and left legs must be set to the same height.
- Do not attempt to move the commode while someone is sitting on it.

### H441 HEIGHT ADJUSTMENT KIT

- The height of H450, H460 and H461 Omega commodes can be adjusted from 53 to 65 cm using a height adjustment kit.
- Remove the four caps from the chair legs and replace them with the four plastic pieces from the kit. Then insert the tubes and adjust the leg height (using the spring buttons).
- A maximum difference of one hole forward or back is permitted. The right and left legs must be set to the same height.

### H750L OMEGA MOBILE COMMODE

#### Locking / transport

- The commode has two brakes at the back. Locking the commode requires at least two brakes to be on. Warning : use in damp conditions can affect the efficiency of the brakes.
- Before transportation, ensure that at least two brakes are on, that the wheels are pointing towards the front and that the footrest is raised. Do not support yourself on the footrest when sitting down or getting up.

#### Moving

- Do not push the commode down slopes or over damaged surfaces. Do not attempt to mount obstacles over 5mm in height.
- Check that no part of the body is outside the commode when passing through doorways (especially near the armrests).

### H407L FOLDING COMMODE

- Unfold the commode and lower the armrests.
- Ensure that the commode is resting on a flat, non-slip surface before use.
- Do not attempt to move the commode while someone is sitting on it.
- The commodes are designed for indoor use only and should not be used as shower seats.

## MAINTENANCE

Clean regularly with a normal bathroom cleaner. Do not use ammonia-based products or bleach on the PVC trim. The bowl can be put through an autoclave (1 hour at 80°). For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

## USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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