AQUATEC®

LAGUNA

LAGUNA, LAGUNA VIP, LAGUNA 24''

SV BRUKSANVISNING

NO BRUKSANVISNING

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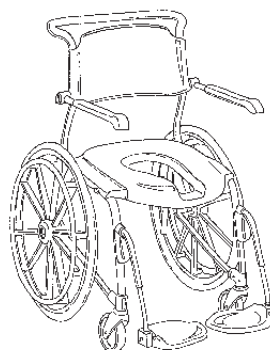
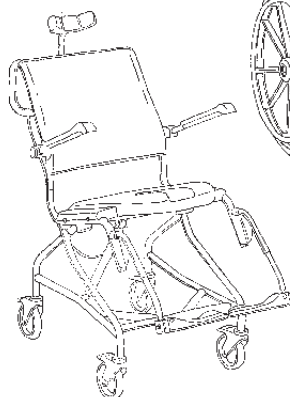
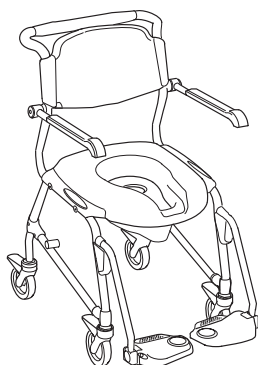
GB OPERATING INSTRUCTIONS

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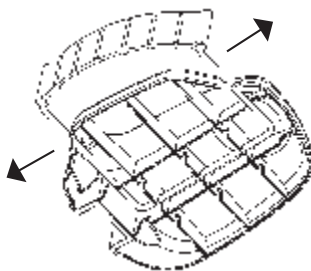
LAGUNA, LAGUNA VIP, LAGUNA 24

GB HEEL LOOP

Standard on AQUATEC LAGUNA VIP, accessory on other models.

- a. Place the steel loop in any of the three possible position on the underside of the footplate. Press.
- b. Extend the steel loop and thread it into the intended tracks in the rear end of the footrest. Thread the heel straps onto the ends of the steel loop. N.B. When the heel loop is placed in the middle or rear position it can in combination with tilting the chair forward result in contact with the front wheel (Tilt-version only).

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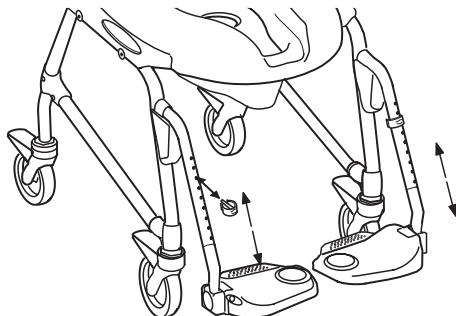
LAGUNA, LAGUNA VIP, LAGUNA 24

GB FOOT RESTS

Height adjustment: Remove the clip from the telescopic tube. Adjust the footrest to required position. Refit the clip.

Adjustment of angle. The hinged footrest makes it possible to angle the footrest, no manual adjustment is required. The resistance can be adjusted using allen key no 6. To facilitate easy transfer in and out of the chair: Lift up the footrest minimum 3 cm and swing to the side. Pull upward to remove the footrest completely. N.B. Do not stand at the footrests. Risk for tilting forward.

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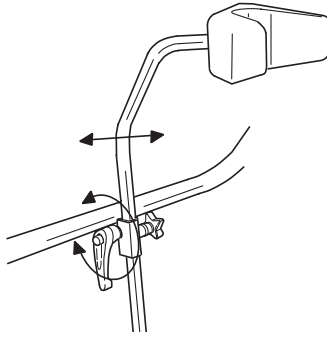
GB HEAD SUPPORT

AQUATEC LAGUNA VIP, only

Height adjustment: 1. Unscrew the black knob.

2. Adjust to required position. 3. Tighten the knob properly.

Adjustment of angle: 1. Loosen the lever in order to adjust the angle of the head support. Please note that the lever is springy. 2. Adjust the neck support to required position.

3. Tighten the lever properly. Do not twist the lever all against the steering handle, to prevent from the risk of getting jammed.

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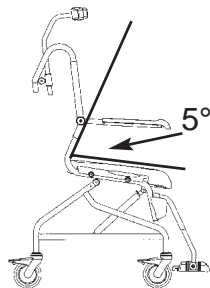
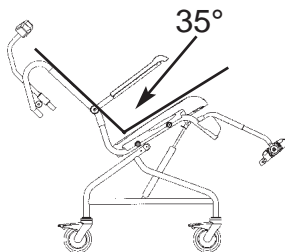
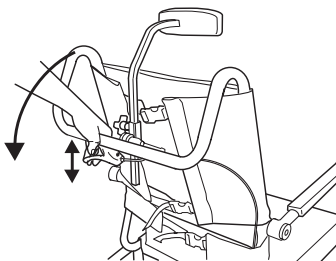
GB TILT MECHANISM

AQUATEC LAGUNA VIP, only

The shower chair is to be tilted when the person is seated only, this due to the fact that the tilting is partly dependent on the persons weight. Pull the handle to tilt the chair backwards or forwards. The patient's legs must not be raised to a level exceeding the footplate level, when the shower chair is placed in a backward-tilted position. Refrain from tilting the chair in the event the chair is being used by people with little or no leg weight as this will alter the balance of the chair.

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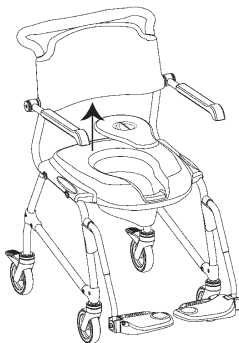
LAGUNA, LAGUNA VIP, LAGUNA 24

GB **SANITARY PAN WITH LID.** (Drawing above)

Remove the sanitary pan by pulling the handle of the pan downwards, then pull out the pan gently. N.B. the lid can be placed onto the pan through the standard seat.

Disassembly of guide rail for sanitary pan

When reversing the chair over the toilet, remove the holder for sanitary pan if necessary, by first removing the seat and then lifting of the metal holder.

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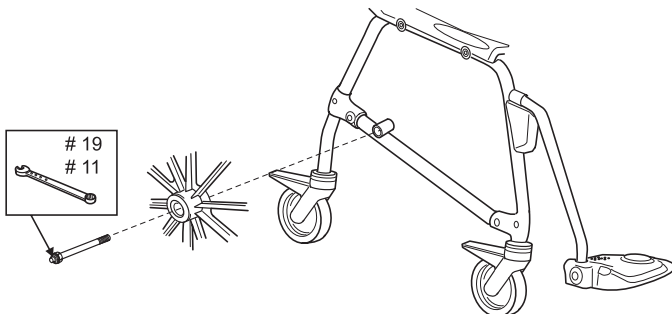
LAGUNA 24, LAGUNA 22

GB ASSEMBLY OF QUICK-RELEASE WHEEL

1. Press the button of the Quick-Release pin. Push the Quick-Release-pin through the wheel and the sleeve of the frame. Release the button.

2. Adjust the Quick-Release screw with the spanners to prevent play.

Using LAGUNA with Quick Release; Wheels Always make sure to lock the rear casters parallel with the self-propelling wheels, to prevent the casters from coming into contact with the self-propelling wheels. Use your foot to push down the red brake flap. The rolling and rotating function of the caster is now blocked. Assembly of the brake, please see page 23.

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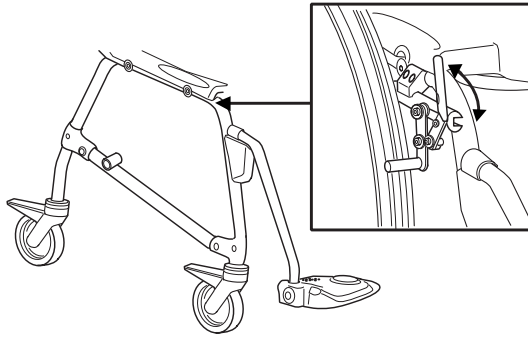
LAGUNA 24, LAGUNA 22

GB ASSEMBLY OF BRAKES.

1. Lift up the lever
2. Assemble the brake on the front part of the horizontal tube, see picture
3. Adjust the distance between brake and wheel. Check brakes.
4. Secure with allen key no. 5
5. Put the seat back on.

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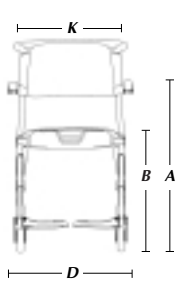
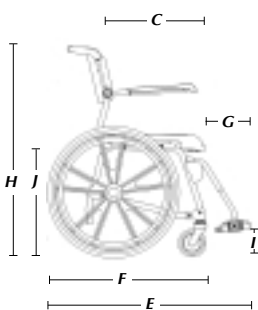
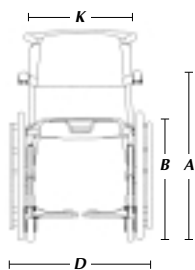
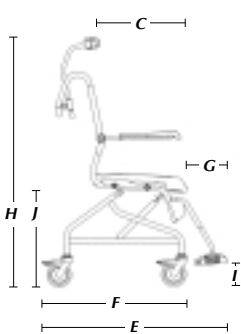
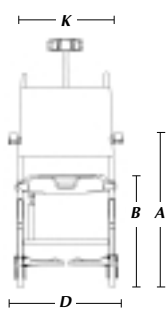
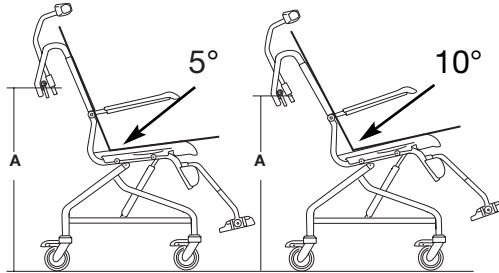
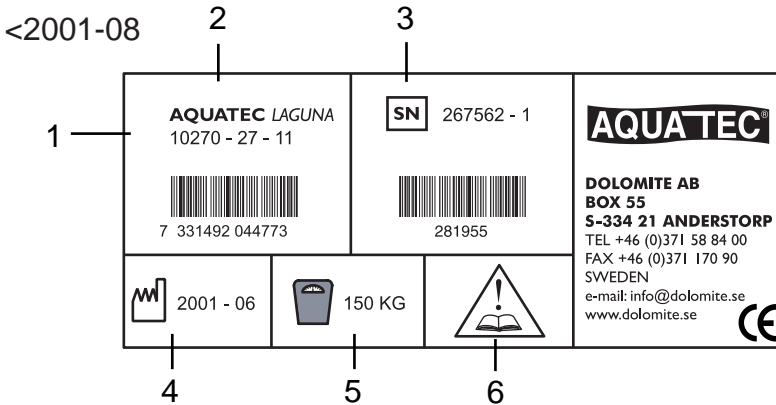
GB

All technical data available on the LAGUNA label:

- | | |
|--------------------------|---------------------------------------|
| 1: Article number | 2: Name of product |
| 3: Serial number | 4: Date of manufacture |
| 5: Maximum user's weight | 6: Attention: see instruction for use |

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GB INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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LAGUNA Mobile Shower/Toilet chair is intended for use as a bedside toilet commode or to be reversed over the toilet and/or as a shower chair. Can also be used in a swimming-pool (not salt water), but only together with an assistant. Minimum two wheels should be locked in position while toileting or showering. The wheels are to be locked the chair is parked. While transporting person, the seat must be placed in horizontal position. *LAGUNA* is to be used on flat surfaces only.

Refrain from tilting the *LAGUNA VIP* in the event the chair is being used by people with little or no leg weight as this will alter the balance of the chair. For optimum stability, position the link wheels forwards as shown in the fig. see page 27.

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the hygiene seat with solvent-free household cleaner. Keep the wheels clean.
- Ensure that screws and adjustments are properly tightened.
- Lubricate the wheels regularly with grease.
- Immediately contact the nearest dealer if a fault should appear. Never use a defective chair. Do not attempt repairs yourself!

Dolomite / Aquatec will not accept responsibility for adaptations, faulty repairs, etc. that are outside our control.

Refrain from tilting the *LAGUNA VIP* in the event the chair is being used by people with little or no leg weight as this will alter the balance of the chair.

Warranty on frame: 3 years. Others: 2 year.

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LAGUNA	150 kg
LAGUNA 24"	150 kg
LAGUNA VIP	125 kg.

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DOLomite AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH