7760
7761
7790
7793
HI40
HI41
HI42
HI43
HI44
HI45
HI46

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Brugsanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

I. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). Our retention bars are guaranteed for two years. Normal wear and inappropriate use are not covered under the guarantee. INVACARE® cannot be held responsible for any injury caused by improper use of its retention bars.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- To ensure your safety, the product must only be installed by a recognised professional.
- The support and retention bars and the handle must be fastened to a sufficiently resistant wall (silico-limestone bricks, or the equivalent). The fastening method (screws, plugs) must be suitable for the type of wall (bricks, hollow partitions, plasterboard, etc.).
- Do not exceed the maximum operating load  or the maximum load  shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

INSTALLATION

For installation, please refer to the drawings at the end of the manual:

- For models HI40, HI41, HI42, H 143, H 144, HI45, see Drawing No. 2.
- For models H 146, see Drawing No. 3.
- For models 7760, 7793 see Drawing No. 2.
- For model 7761, see Drawing No. 4.
- For model 7790, see Drawing No. 3.
- The bars can be fastened freely. Drawing No.1 shows the normal fixing position for a short retention bar.
- Position the bar in the position desired and mark the location of the fixing holes.
- Drill and fasten using the appropriate plugs for the type of support.
- Warning: the stainless steel washers (supplied with the product) must be inserted in the spaces provided for them on the handle.
- On a brick wall, use the 4 screws (1), the 4 plugs (3) and the 4 washers provided. On any other type of wall, use appropriate supports.
- We recommend that you use a screwdriver with a sufficiently long end (> 10 cm) to avoid damaging the handle.

MAINTENANCE

Clean regularly with a normal household cleaner. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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