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# Toilettenstuhl »Fahrbar«

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## User Instructions

# Commode chair »mobile« with swivelling armrests and footrests

Item No. 17502 100

Item No. 17502 200

Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

The commode chair enables greater independence when using the toilet and improves the personal hygiene associated with this, in particular by compensating for restricted or absent movement functions in the musculoskeletal system or supporting ligaments. It is suitable for patients with a body weight of max. 120 kg.

### Safety Instructions

- Before use, always check whether the commode chair is in a perfect condition and ensure that all connections are firmly in place!
- Please ask your specialist dealer if you are uncertain about anything or have any questions concerning assembly or care!
- Do not exceed the specified maximum weight of 120 kg for users!
- Please ensure that the commode chair corresponds to your requirements. If in doubt, ask your doctor or specialist dealer!
- Missing, damaged or worn parts may only be replaced by the specialist dealer!
- Do not use, leave or store the commode chair outdoors!
- Only use the commode chair on horizontal, level and solid ground!
- Avoid leaning too far forwards and abrupt movements backwards!
- The brakes must be locked when using in conjunction with a toilet bowl!
- Do not at any time sit on the arm rests!
- Never put your weight onto the footrests when sitting down or getting up!

### Field of application

The use of the commode chair is indicated for patients lacking truncal control as well as those with balance problems and considerable loss of function in the lower extremities, patients with hip and spinal column stiffening, patients without hands, tetraplegics or those with arm damage and significant impairment of the gripping function.

### Scope of supply

The commode chair is supplied partially preassembled in the packaging. Please check that the contents are complete:

- 1 instructions for use
- 1 body frame with 2 castors, 2 braking castors and 2 adjusting screws
- 1 backrest with 2 padded armrests and 2 snaps
- 2 footrests
- 1 ABS toilet seat
- 1 commode bucket with lid and handle
- 1 seat cushion
- 1 calf strap with Velcro fastener


## Assembly

To be able to fully assemble the commode chair, position the body frame in front of you and lock the braking castors, so that the commode chair cannot roll away during assembly.

### Mounting the back and armrests

1. Remove the backrest from the packaging. Insert the rod ends of the backrest into the rod ends of the body frame until they lock into place. Ensure that the inside of the backrest points towards the centre of the chair.
2. Turn the two setting screws on the rear of the chair clockwise and tighten.
3. The armrests should lock into place themselves if you swivel them upwards in one short movement.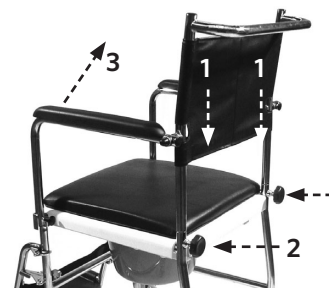

### Mounting the footrests


Insert the footrest bars into the mounting pins intended for this on the main frame. Swivel the footrest bars towards the centre. The locking mechanism automatically snaps into place. You can now fold down the foot plates. Finally attach the calf strap to its holder on the footrest bar.

### Setting the footrest height

For this you require an open-ended or ring spanner SW10.

Set the height of the footrests so that your thigh is horizontal if you have your feet on the footrests while sitting.

1. Loosen the locking screw on the footrest rod.
2. Push the footrest to the required height. The rod may not be pulled out beyond the rod marking! Tighten the locking screw again.

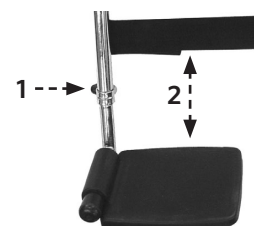### Assembling the seat and commode bucket

1. Take the white seat and place it onto the main frame from above. Ensure that the seat plate is aligned correctly: The closed, round edge of the seat should point forwards, the commode bucket guide on the seat underside should point to the rear.
2. You can push the commode bucket into the guide from the rear below the seat face, with or without its lid.

Finally you can place the padded seat cover on the toilet seat to increase comfort when sitting normally.

## Usage

### Sitting down and getting up


#### Important! Never put your weight onto the footrests when sitting down or getting up!

- Position the commode chair and seat which you wish to sit on or move to as close beside each other as possible.
- Secure both sitting aids by locking the brakes.
- Swivel the side pieces and leg supports out of the way.
- Now slide sideways onto the seat. Please ensure that you sit as far back in the seat as possible.
- Swivel the arm and leg supports back into their initial position.

### Brakes

If the commode chair is not to be moved, lock the brakes on the two braking castors.

1. The braking castors are locked by simply operating the brake handle with your feet.
2. To release the brakes, press the rear part of the brake lever back into its initial position using your foot.

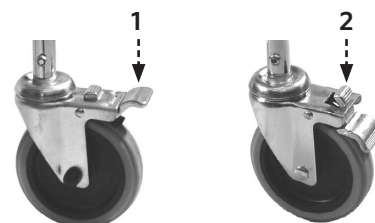Lock unbraked castor      Release braked castor

### Commode chair bucket

Before using the commode chair bucket, remove the bucket lid. Now push the commode chair bucket into its guide rails from the rear below the seat. Push the bucket the whole way until it noticeably locks into place and is thus secured against falling out.

Practical tip: Before use, cover the commode chair bucket with some toilet paper or add some water to it. This makes it easier to clean later.

### Armrests

You can swivel the armrests backwards to make sitting down or getting up easier.

1. Pull on the snap on the armrest
2. and swivel the armrest downwards out of the way. If you pull the armrest upwards, the snap automatically locks into place again and the armrest is secured.

### Footrests

The footrests can be folded up and swivelled to the side.

To swivel, pull the lever inwards and the footrest is now adjustable.

## Cleaning/Desinfection

The commode chair should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM-/VAH-listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

## Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines.

## Maintenance

The commode chair is maintenance-free. Simply check regularly and according to maintenance plan that the entire commode chair as well as its components and connections are undamaged.

## Material

- Frame: steel chrome plated (17502 100) or powder coated (17502 200)
- Toilet seat: PP
- Padded seat/backrest: PVC
- Armrest: PU
- Footrest: PP

## Disposal

Please ask your specialist dealer for the regional disposal regulations.

## Technical data

- Dimensions: 87 x 57 x 93 cm (L x W x H)
- Empty weight: 13,3 kg
- Seat dimensions without seating cushion: 49 x 45 x 45 cm (H x W x D).
- Armrest: 22 x 5 x 44 cm (H x W x L).
- Backrest height: 41 cm
- Backrest angle to the rear: 7°
- Max. load bearing capacity: 120 kg
- Castor dimensions: 13 x 3 cm (Ø x width)

## Warranty

The statutory provisions for warranty apply (24 months from the date of purchase). This warranty does not apply if the product defect can be traced back to improper treatment and/or disregard for the intended use as described in these instructions.

Wearing parts are excluded from the warranty. Only original spare parts must be used.

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