7762
7763
7764
7765
7766
7767
7768
7770
7772
7773
7780
H335
H345

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Bruksanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

I. FRANCAIS

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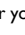
2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer. Our retention bars are guaranteed for two years. Normal wear and inappropriate use are not covered under the guarantee. INVACARE® cannot be held responsible for any injury caused by improper use of its retention bars.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load  or the maximum load  shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

7762 / H335 (SUPPORT BARS)

Introduction

The support bar is shown in the minimum, basic version without a supporting foot (Figure A).

Defining the installation height

With the user standing upright, without shoes, measure the height of the thumb's upper joint. This height corresponds to the support bar's highest point (Fig. A1).

Installation

- Inserting the 2° wedge (Fig. A6).
- Mark out the location of the holes on the wall (2 above and 2 below outside) taking into account the previously measured height; position the socket for the support bar accordingly (Fig. A5).
- Drill and insert 4 plugs appropriate for the type of wall. Screw (Fig. A3).
- Snap the cowl onto the socket (Fig. A2).
- Insert the wedge from below (Fig. A4).

Adjusting the height

The support bar is supplied with a 2° wedge (Figs. A4 and A6). To adjust the height of the support bar, use option 7768 (3 wedges of 0°, 2° and 4° resp.). Proceed as follows:

- Remove the 2° wedge.
- To lower by 1cm, insert the 0° wedge.
- To raise by 1cm, insert the 4° wedge.

Moving the support bar

To raise: lift the support bar from the horizontal position to its maximum vertical position. It locks automatically in this position.

To lower: lift the support bar approx. 1cm to unlock it and hold it as it drops down to the horizontal position.

7763 (CONSOLE)

Introduction

This option, when sealed to the floor, enables a support bar (7762, 7770, 7772 or 7773) to be fastened in place when the wall is unsuitable for this.

Installation

- Position the socket (Fig. B3). The plate resting on the floor must have its longest side under the support bar (Fig. B6).
- Prick out the location of the 4 screws. Drill and insert 4 plugs appropriate to the type of wall. Screw (Fig. B5).
- Position the plate – with the rounded section towards the bottom (Fig. B2) and the support bar (Fig. B7). Fasten the plate and the bar with the two clamps (Fig. B1), washers and nuts supplied but without tightening them (to allow for subsequent adjustment).
- With the user standing upright, without shoes, measure the height of the upper joint of the user's thumb. Tighten the nuts (Fig. B4) on the two clamps (Fig. B1).
- Check that the whole assembly is firm.

OPTIONS

The basic 7762 / H335 version can be fitted with the following options:

- A retention foot (7764 / H335) increasing the support bar's load capacity (Fig. C).
- A toilet paper holder (7765) mounted on the support bar (Fig. D9).
- A soft, comfortable armrest (7766) glued to the end of the upper support bar (Fig. E3).
- An aesthetic cover (7767) designed to hide the lower tube (Fig. D2).

7764 / H345 (SUPPORT FOOT)

Installation

- Remove the socket's cowl (Fig. A2).
- Remove the end cap, if fitted (Fig. C3) or the cover (Fig. D2) on the lower tube (Fig. C4).
- Install the hinge element (Fig. C2) by sliding it onto the lower tube. If necessary, use a plastic mallet. Fix with two sets of screws and nuts (Fig. C1) and position the two covers.
- Remove the lower left-hand fixing screw from the socket (Fig. C7).
- Position the guide (Fig. C10), with the end with the wall hinge (Fig. C6) towards the support wall or bracket on the external side of the support bar.
- Fasten the wall bracket (Fig. C6) using the socket's lower, left-hand side fixing screw (Fig. C7).
- Insert the other end of the guide (Fig. C10) into the oblong hole in the support foot (Fig. C9).
- Firmly insert the other end of the guide's retention cowl (Fig. C8).
- Check that the whole assembly is firm and operates correctly

7765 (TOILET PAPER HOLDER)

Installation

- If fitted, remove the end cap (Fig. C3) or the cover (Fig. D2) or the hinge (Fig. C11) from the lower support bar.
- Remove the cover, using a screwdriver (Fig. D8 or C11).
- Insert the toilet roll support (Fig. D9) into the fixing holder (Fig. D2) or the hinge (Fig. C11).
- Position the intermediate socket (Fig. D3) and lock the assembly in place with the locking ring (Fig. D6).
- Install the assembly (Figs. D2, 6, 7 and 9) by sliding it onto the lower tube (Fig. D5). If necessary, use a plastic mallet. Fasten with a screw (Fig. D1) and a nut (Fig. D7) and position the cover (Fig. D7).
- Check that the whole assembly is firm.

7766 (ARMREST)

Installation

- If fitted, remove the black end cap (Fig. E5) on the support bar's upper tube.
- Clean the parts to be glued together with acetone before applying the glue (Fig. E2).
- Apply the whole tube of glue to the inside face of the armrest (Fig. E1) and then place this directly onto the upper part of the support bar, taking care to lay it symmetrically (Fig. E3).
- Press it down for approximately 30 seconds. Full adhesion is only obtained after approximately 24 hours. INVACARE® is not responsible for any damage caused by improper fixing or inappropriate use.

7767 (PLASTIC COVER)

Installation

- If fitted, remove the end cap (Fig. C3) from the support bar's lower tube.
- Thread the plastic cover (Fig. D2) over the lower tube. If necessary, use a plastic mallet to insert the cover completely.
- Insert the screw (Fig. D1) and the nut (Fig. D7) and tighten. Position the cowls.

MAINTENANCE

Regularly clean the support bar and its accessories with a conventional, non-aggressive household cleaner. To disinfect in a hospital environment, follow the Hygiene Specialist's recommendations.

TECHNICAL CHARACTERISTICS

- Metal tubes covered with shock-resistant Rilsan.
- Hinge points in stainless or tempered steel.
- Polyurethane armrest.
- Support height-adjustable to suit the individual.
- 50 and 100cm lengths available.
- Weight of the basic 7762 / H335 version: approx. 4 kg.

MAXIMUM USER WEIGHT

See table at the end of this document.

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