**C500**  
**C510**  
**C520**  
**C530**  
**C540**  
**C570**  
**C254**  
**C254H**  
**C257**  
**C258**

**F** 1. Guide utilisateur  
**GB** 2. User's manual  
**D** 3. Gebrauchsanweisung  
**E** 4. Manual del usuario  
**I** 5. Manuale dell'utente  
**NL** 6. Gebruiksaanwijzing  
**P** 7. Guia do utilizador

CE

## 1. FRANCAIS

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XS	Extra Small	
	Noir	
S	Small	Jaune
M	Medium	Vert
L	Large	Rouge

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## 2. ENGLISH

You have just purchased an Invacare® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Do not leave or store the product out of doors.

**These operating instructions do not constitute a training manual.**

Before carrying out any transfer, ensure that you are appropriately trained to operate lifting equipment and slings so that you can transfer the patient in complete safety and comfort. If you require any further information please contact your dealer or a health professional.

### OPERATING INSTRUCTIONS

- Invacare® slings are designed to be used with Invacare® lifting equipment (lifters and hoists) but can also be used with other manufacturer's lifting equipment. The sling's maximum load may differ from the maximum load for the lifting equipment you will be using: always check that the patient's weight is less than the maximum loads indicated on the sling and the lifting equipment.
- Most Invacare® slings are available in 4 sizes, identified by the following colours:

XS	Extra Small	
	Black	
S	Small	Yellow
M	Medium	Green
L	Large	Red

- Check that the sling is in good condition before use (that there are no holes or tears, that the stitches are not broken, etc.).
- The slings are attached to the lifting bar by straps with various loops that enable their length to be adjusted. With practice, you will be able to adjust the straps so that the patient is in a sitting or semi-reclining position. As an indication, a short setting for the shoulders and a long setting for the legs provide a sitting position. Colour codes help you to adjust the straps to the correct length. Warning: the settings for the different types of strap (shoulder, centre, leg) can differ but each set of straps must be adjusted to the same setting on both sides of the patient, i.e. if the shoulder strap is set on the red loop on the right-hand side, it must also be set on the red loop on the left-hand side.

- When positioning the sling, always ensure that the patient's arms remain outside the sling.
  - When all the straps are correctly attached to the lifter bar, gently raise the patient using the lifter until the straps are taut. Ensure that none of the straps are twisted before continuing to lift.
  - The C500 sling should only be used for patients who have good torso control.
  - The sling material used on C254, C254H, C257 and C258 is inflammable. Do not use these slings near a naked flame or while smoking.
- For any further information, follow the instructions in the user's guide for the lifting equipment you are using with an Invacare® sling.

#### MAINTENANCE – USER'S MAXIMUM WEIGHT

To avoid the risk of infection, the sling should always be washed before using it again for a new patient.

All Invacare® slings are machine washable.

On the C500 model, always attach the velcro before washing to avoid deterioration.

Machine washing (°C)	(kg)
C500 / C510 / C520 / C530 / C540 / C570	200
C254 / C254H / C257 / C258	170

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