**P429BE
P429BKE
P429E
P429KE
P452E
P550E
P472**

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Brugsanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

I. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

OPENING THE FRAME

The frame should always be locked in the unfolded position during use.

2 and 3 wheel rollators (P550E, P429E, P429KE, P429BE, P429BKE)

- Press downwards to unfold and lock the opening mechanism in position.

4 wheel rollators (P452E, P472)

- On initial assembly, remove the plastic crossbar between the 2 rear wheels.
- Unfold the rollator and lock the mechanism in position using the peg on the right-hand side.

FITTING THE HANDLES

- If fitted, remove the caps from the ends of the side tubes.
- P429E, P429KE, P429BE, P429BKE, P452E
Slide the handle tubes into the side uprights and tighten the handles. Fit the right-hand handle on the right and the left-hand handle on the left with the flat sections facing inwards.
- Tighten the tube tightening levers and position them in line with the tube to avoid any inconvenience during use. Never set the handles above the safety marks (the hatched part of the tube).
- The maximum angle of opening for the handles () is 30° to the rollator's main axis.
- P472
Slide the handle tubes into the side uprights and tighten the handles. Fit the right-hand handle on the right and the left-hand handle on the left with the flat sections facing inwards.
- Lock the handle by tightening the 2 side grips.

USE OF THE BRAKES

P429E, P429KE

- Press on the handles to brake. The tension on the brake can be adjusted to the user's weight and capabilities using the setting screws on the rear wheels.
 - Loosening the setting screw reduces the braking effort required.
 - Tightening the setting screw increases the braking effort required.

P429BE, P429BKE, P452E, P472

- When walking, the brakes are used like those on a bicycle. A cable adjuster fitted on the brake lever adjusts the tension of the brake and adapts the braking level to suit the user's capabilities.
 - □ Loosening the adjuster eases braking and allows the brake lever to move more freely.
 - □ Tightening the adjuster makes braking harder and restricts movement of the brake lever.
- The 2 parking brakes must be locked before sitting on the seat. On P429BE, P429BKE and P472, the parking brakes are locked by tightening the brake and pressing the small

chrome knob on the outside of the brake. On P452E and P472, the parking brakes are engaged by pressing the handles downwards.

- To release the brakes, pull the brake lever upwards.

FOLDING

2 and 3 wheel rollators (P550E, P429E, P429KE, P429BE, P429BKE):

- Pull the crossbar on the folding mechanism upwards.

4 wheel rollators:

- P452E lateral locking system: unlock the lever and pull it back and up.
- P472 includes a side locking system: unlock the lever on the right hand side and pull the metal crossbar of the foldaway system upwards.

MAINTENANCE

Use a soft, damp cloth to clean away dirt and dust. Use soap and water to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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